

POLYSOFT - *EXTREME*

Extreme Performance

Extreme Resistance

Extreme Resistance to Breakdown

When in contact with correctly diluted disinfectants & cleaners based on:



POLYSOFT - *Extreme* Features

- ✓ Sanitized® giving optimal protection against bacteria, microbes, fungi and dust mites
- ✓ Effective against MRSA
- ✓ Resistant to blood, urine, fat and oil
- ✓ Non – allergenic
- ✓ Washable to 95°C
- ✓ Waterproof tested to ISO 811 > 3000mm
- ✓ Extreme hydrolysis resistance (tested to ISO 1419)
- ✓ Multi-directional elasticity for excellent pressure care
- ✓ Gentle to the skin
- ✓ European manufacture
- ✓ Superior Longevity



Approved Cleaners for use with Polysoft – *EXTREME*

Cleaning Product	Concentration
Actimax (Worldwide)	According to the manufacturer's specifications.
Arjo Disinfectant	According to the manufacturer's specifications.
Clinell approved products: Clinell Hand & Surface Sanitizer Disinfectant Clinell Universal Sanitising Wipe Clinell Detergent Wipes Clinell Sporidical Wipes (Global Medics, Australia & New Zealand)	According to the manufacturer's specifications.
Divermite D2 (Johnson Diversey, Australia)	1 part Divermite D2 to 150 parts water (5ml per 750ml spray bottle)
Divermite D4 (Johnson Diversey, Australia)	1 part Divermite D4 to 100 parts water (5ml per 750ml spray bottle)
Divermite D10 (Johnson Diversey, Australia)	1 part Divermite D10 to 50 parts water (10ml per 500ml spray bottle)
Divercleanse	125ml:4 Litres of Water
Eco Pine (Velocity Chemicals, America)	5% Ecopine (1 part Ecopine to 20 parts water (50ml per litre)
Isopropyl Alcohol 70%	
Jumbo Liv-Wipes	According to the manufacturer's specifications.
Leggeside HB (Legge Systems, America)	According to the manufacturer's specifications.
Mediwipes Disinfection (Sulco Limited)	According to the manufacturer's specifications.
Pal Detergent Wipes	According to the manufacturer's specifications.
Pal Disinfectant Wipes (Globalmedics)	According to the manufacturer's specifications.
PureGreen 24 (Pure Green 24, America)	According to the manufacturer's specifications.
Sanicare TBX	According to the manufacturer's specifications.
Sanicare Disinfectant Wipes (Buckeye International)	According to the manufacturer's specifications.
Sodium Hypochlorite	13000ppm (Approved Colours)
Spring Clean (Eco Lab, Australia)	5% Spring Clean (1 part Spring clean to 20 parts water (50ml per litre)
Sterrimatt (pure) Foam	Ready to use
Super Sani Cloth Germicidal Disposable Wipes (PDL, America)	According to the manufacturer's specifications.

Surface Sanitising Wipes Neutral Detergent Wipes (Reynard Health Supplies, Australia, New Zealand & Europe)	According to the manufacturer's specifications.
Terminator One Step Disinfectant (Buckeye Intl)	According to the manufacturer's specifications.
Trigene (Bio Strategy, Australia)	According to the manufacturer's specifications.
Trix (soap)	Ready to use
Tuffie 5 Cleaning Wipes	According to the manufacturer's specifications.
Viraclean (Whiteley Medical, Australia)	According to the manufacturer's specifications.
Virkon (Dupont, Worldwide)	5% Virkon (1 part Virkon to 20 parts water (50ml per litre))
<p><u>Polysoft – EXTREME only.</u></p> <p>Generally, Polysoft <i>EXTREME</i> resists a broad spectrum of cleaners/disinfectants as commonly used in hospitals and aged care facilities. Typical surface wiping can be performed using cleaners/disinfectants based on the following chemicals diluted as per the manufacturers recommendation's:</p> <p>* Chlorine *Alcohol *Peroxide *Quaternary Ammonium *Chloramine *Phenol</p> <p>NB: Up to 13000ppm chlorine and only for approved colours.</p>	
<p>IMPORTANT FOR ALL CLEANING PRODUCTS</p> <p>Wipe off cleaning product residue with a clean, soft cloth and rinse with fresh warm water immediately after use.</p> <p>If particular cleaner/disinfectant is not listed we advise to test the particular surface disinfectant in advance or to contact us in case of any doubts.</p>	

POLYSOFT - *EXTREME* HD FRC

SPECIFICATIONS

WEIGHT:	± 230 g/m ²
COMPOSTION:	59% Polyamide 41% Polyurethane (Antimicrobial treated)
WIDTH:	120cm – 220cm
TENSILE STRENGTH:	ISO 13934-1 Warp: > 600 N Weft: > 250 N
TEARING STRENGTH:	ISO 13937-1 Warp: > 35 N Weft: > 20 N
BREAKING EXTENSION:	ISO 13934-1 Warp: > 80 % Weft: > 170 %
WATERPROOFNESS:	ISO 811 > 3000 mm
BREATHABILITY:	DIN 53122 ± 400 g/m ² /24hr
FIRE STANDARD:	BS 6807 ignition source 5 BS 7175 ignition source 5



Washable to 95°C



- Up to 13000ppm chlorine
- Only for approved colours



Can be tumbled dry – if any other fabrics that are sewn with this are approved for doing so.
Set on low temperature and remove fabric from dryer immediately when dry



Dry Cleanable – using any solvent except Trichloroethylene



Do not iron